


COMPONENTS




PRODUCT CODE
AM3000

-  **A** Foot Plate x 1
-  **B** Leg Pad x 1
-  **C** Lower Leg Support x 1
-  **D** Leg Strap x 3
-  **E** Foot Strap x 2
-  **F** Table Strap x 2

UNBOXING

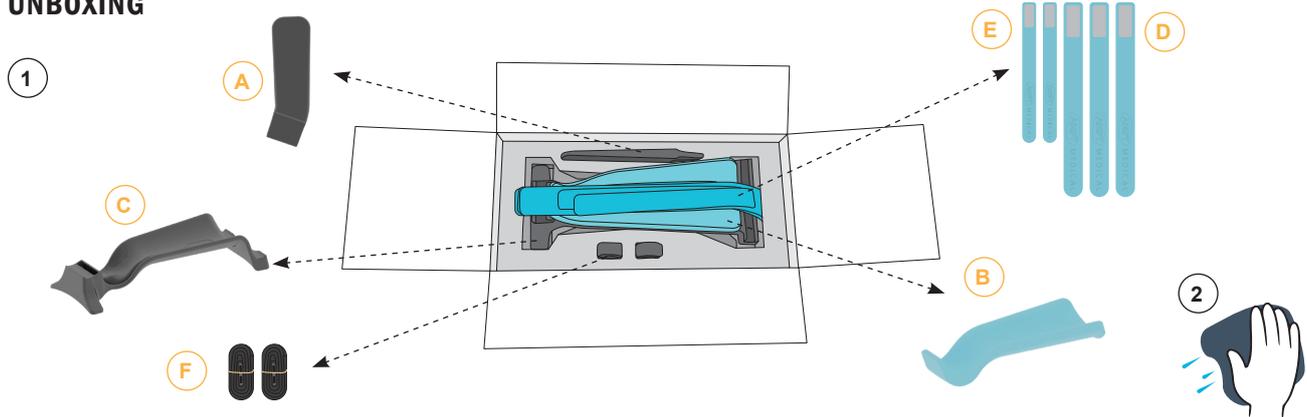

ASSEMBLY

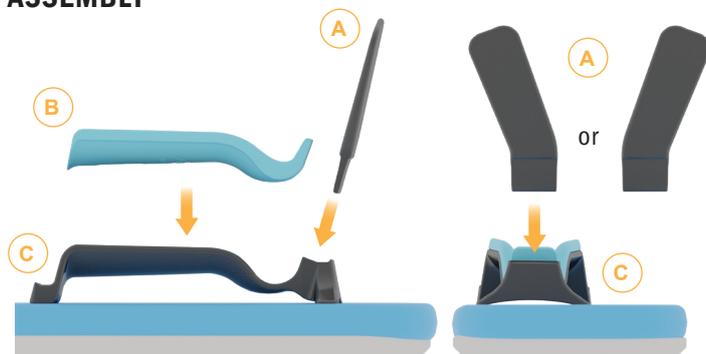

-  For online training visit:
[AdeptMedicalTraining.com](https://www.adeptmedical.com/adeptmedicaltraining)
-  For service and repairs visit:
[AdeptMedical.com/repairs](https://www.adeptmedical.com/repairs)
-  For Approved Disinfectants PDF visit:
[AdeptMedicalTraining.com/product-cleaning](https://www.adeptmedical.com/product-cleaning)

CLEANING



Disinfect and wipe clean before/after each use. **DO NOT USE** any other sterilisation methods.

MAXIMUM
PATIENT WEIGHT



MANUFACTURER

Adept Medical Ltd
2-6 McDonald St, Morningside
Auckland 1025, New Zealand
www.adeptmedical.co.nz

EC REPRESENTATIVE

Emergo Europe
Prinsessegracht 20
2514 AP The Hague
The Netherlands




SETUP | BEFORE PATIENT


Buckle Orientation




Right Leg

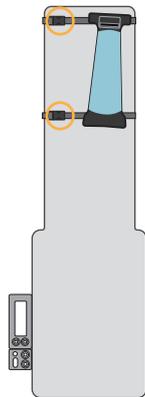

Left Leg

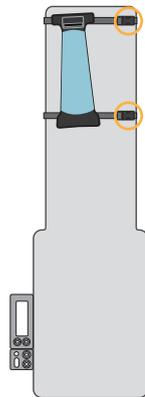

Strap Placement

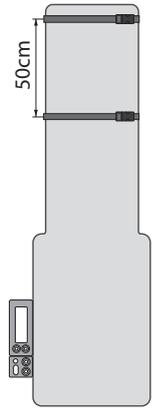


1



2




3 Place the Patient on the Table

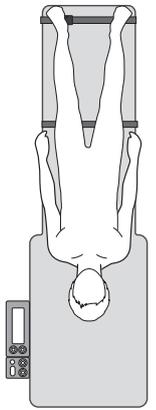

SETUP | PATIENT LEG POSITIONING

1

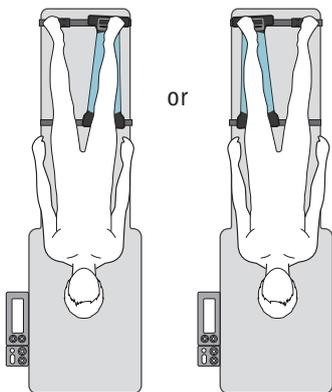

2



3




TIGHTENING TABLE STRAPS

1 Tighten Straps Firmly to Immobilise the Support.



2

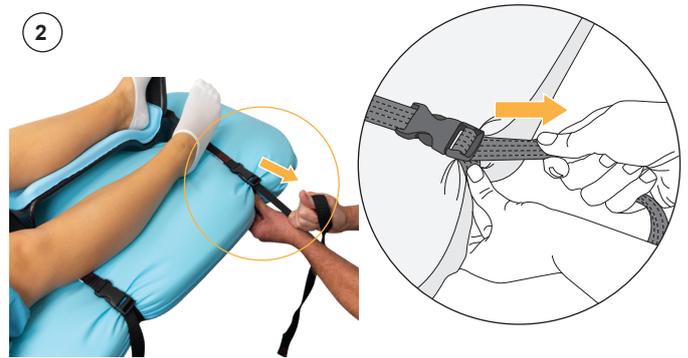

OPTIONAL PADDING (If Required)


Cover Buckles




Supporting the Knee



APPLYING THE STRAPS | LEG STRAPS

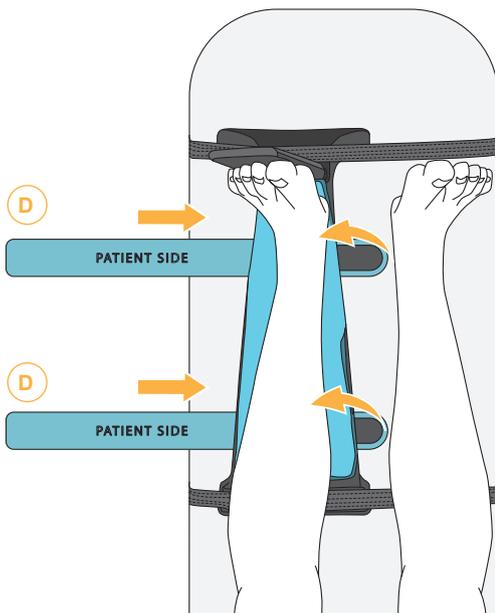

APPLYING THE STRAPS | FOOT STRAP

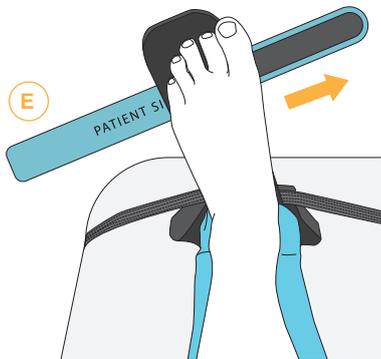

REMOVING THE FOOT PLATE



FOOTPLATE POSITIONING OPTIONS

Exorotation



Endorotation



or

ACCESSING THE FOOT

